

The Connect version 8.02.01 with following enhancements and changes

- Application could report GUID issues on start-up and forced to exit the program.
- When sent measured data back to IOLMaster 500, the numeric format should use US type of format not the local culture format.
- Fixed an issue which B-Scan/UBM image snapshots won't show in the "Reports" menu with 4:3 monitor (1280x1024 resolution).
- Fixed the issue of B-Scan/UBM mixing image snapshots with different aspect ratio (4:3 and 16:9).
- Changed required patient info (last name, first name, and MRN) follow the folder name options in the Setup menu.
- Fixed the issue that "A-Const" background color in IOL calculation table was not changed even user modified it manually.
- Implemented functions to save a copy of A-Scan and Pachymeter reports in the folder of the exam (same as B-Scan reports).
- Fixed a bug that could not send A-Scan data to DICOM server if "Setup->Physician->Department" was set and no B-Scan data was created.
- Fixed a bug that exam creating date was not set if user did not press "New Exam" button.